

HITACHI 717 **β -Hydroxybutyrate LiquiColor[®] Test (Endpoint)**

Chemistry settings
Temperature: 37°C

PROGRAM PARAMETERS

TEST	(BHY)
ASSAY CODE	(2 POINT): (24) - (50)
SAMPLE VOLUME (μL)	(8) (4)
R1 (RGT A) VOLUME (μL)	(300) (20) (NO)
R2 (RGT B) VOLUME (μL)	(50) (20) (NO)
WAVELENGTH (NM)	() (505)
CALIB. METHOD	(K-Factor) (0) (0)**
STD 1 CONC. - POS.	(0.0) - (1)
STD 2 CONC. - POS.	(0) - (0)
STD 3 CONC. - POS.	(0) - (0)
STD 4 CONC. - POS.	(0) - (0)
STD 5 CONC. - POS.	(0) - (0)
STD 6 CONC. - POS.	(0) - (0)
SD LIMIT	(0.1)
DUPLICATIVE LIMIT	(100)
SENSITIVITY LIMIT	<user defined>
ABS. LIMIT (INC/DEC)	(0) (INCREASE)
PROZONE LIMIT	(0) (LOWER)
EXPECTED VALUE (MG/L)	(0) - (0.3)
TECH LIMIT	(0.1) - (4.5)
INSTRUMENT FACTOR	(1.0)

**K Factor: Refer to Monitor, JOB3, Calibration List.

Order Information

Cat. No.
2440-058

Kit Size
R1: 1 x 50 mL
R2: 1 x 8.5 mL
Std: 1 x 3 mL

Notes

1. Refer to the package insert of the β -Hydroxybutyrate LiquiColor[®] Test for detailed information about the test on the following:

Clinical Relevance
Method and Principle
Composition and Stability of the Reagents
Specimens
Calibrators and Controls
Performance Characteristics regarding
- Measuring Range
- Specificity/Interferences
- Sensitivity/Limit of Detection
- Precision (Reproducibility, Repeatability)
Reference Ranges
Literature

2. Manufactured by:
Stanbio Laboratory
1261 North Main Street
Boerne, Texas 78006, USA
e-mail: stanbio@stanbio.com