

# True® 20 & True® 20 Plus One-Step Pregnancy Test



Easy-to-read results

Convenient room  
temperature storage

Rapid results

Urine and serum  
samples

## True® 20 & True® 20 Plus

Truly Accurate, Truly Economical,  
Truly Exceptional

CLIA '88  
WAIVED



**STANBIO**  
LABORATORY

An EKF Diagnostics Company

# True® 20 & True® 20 Plus One-Step Pregnancy Test

## True® 20 & True® 20 Plus

Everything you need in a convenient pouch (including dropper).

### Exceptional Readability

Quick clearing background ensures easy-to-read results even with low-level specimens

### Fast

Positive results in as little as 10 seconds\*

### Sensitive

Detects 20 mIU/mL hCG or greater in both urine and serum specimens

### Convenient

Room temperature storage

### Reliable, Specific, and Accurate



Specific mono and polyclonal antibodies ensure early pregnancy detection while preventing cross reactivity with other hormones present in patient samples



\* The test is positive if two colored lines appear. One colored line will appear at the Specimen Zone and one at the Control Zone.

### General test procedure

- Remove **True® 20 / True® 20 Plus** device and disposable dropper from foil pouch
- Label the test device with patient or control identification
- Dispense two drops of the specimen into the round sample well
- Read result as positive or negative

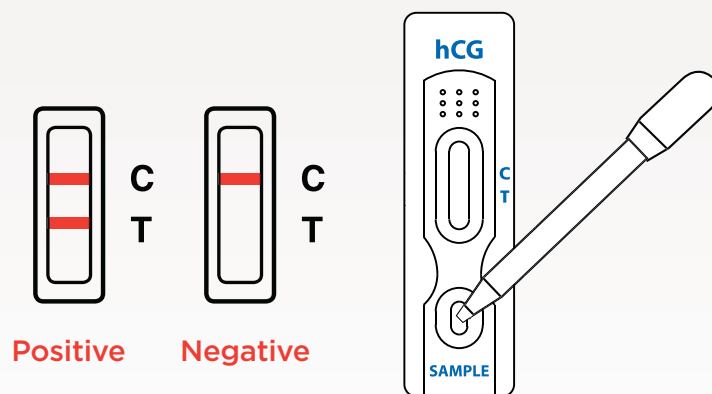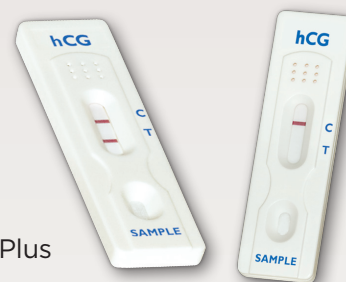

Reference No.

1430-050 True® 20 hCG (Urine) (50 Tests)

1440-050 True® 20 Plus hCG (Serum/Urine) (50 Tests)

Distributed by

**For over 50 years Stanbio Laboratory has been the leader in clinical diagnostics and is committed to provide only the highest quality products.**

### Stanbio Laboratory

1261 North Main Street  
Boerne, Texas  
78006 USA  
866.782.6246  
(U.S. Toll Free)  
Tel: 830.249.0772  
Fax: 830.249.0851  
stanbio@stanbio.com

ISO 13485:  
2003 Certified



CODE\* : 81025 (Urine)  
84703 (Serum)



**STANBIO**  
LABORATORY

An EKF Diagnostics Company

\* Exemplar code provided. Refer to regional CMS fee schedule for relevant codes and reimbursement rates.

RSS0012.01 Eff. Date: 10/12/12 DAR #: 12.1883 ©2012 Stanbio Laboratory